

ReNerve^{LTD}



**Lyophilized
Amniotic Allografts**

**Protective barrier for
complex wounds**

Patients with non-healing wounds face challenges that limit healing, leading to chronic defects and increasing risk of morbidity, amputation, and mortality.¹ Ongoing care for non-healing wounds also has an economic cost estimated in the billions annually.²

Empliq™ amniotic allografts act as a barrier and protect the wound bed during acute and chronic wound closures.



Retains native components of amniotic membrane



Easy to handle, orient, and apply



Terminally sterilized



No rehydration required



Shelf-stable



Multiple size options


The Empliq Range

Empliq allografts are intended for homologous use as a barrier, forming a protective cover over acute and chronic wounds.




Empliq Placental

Proprietary process retaining chorion and amnion

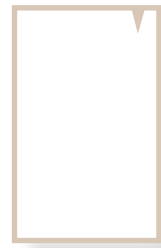

Empliq Amnio-Lite

Amnion only, no chorion

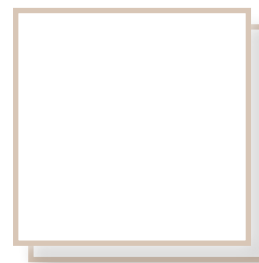

Empliq Amnio-Duo

A double-layered amniotic membrane with improved handling characteristics

Empliq Amnio-Lite and Empliq Placental have two distinct sides: an epithelial side and a stromal side. The epithelial side is smooth while the stromal side is dull.

No orientation requirements, both sides are identical.

Application Guide

1

Select appropriate size and remove from packaging.

2

Apply stromal-side down against the wound bed, and trim excess away or fit to the contours of the wound as required. *(No specific orientation is required for Empliq Amnio-Duo.)*

3

Cover with a non-adherent dressing, followed by moistened gauze to fill, but not pack the wound.

4

Change dressings and reapply as clinically indicated.



For complete application instructions and other product details, please refer to the Instructions for Use

ReNerve^{LTD}



Ordering Information

Lyophilized Amniotic Allografts



Empliq Placental

Lyophilized Placental Membrane

Code	Size (cm) (length x width)
EM-PL2020	2 x 2
EM-PL4040	4 x 4
EM-PL4060	4 x 6



Empliq Amnio-Duo

Lyophilized Double Amniotic Membrane

Code	Size (cm) (length x width)
EM-AD2020	2 x 2
EM-AD4040	4 x 4
EM-AD4060	4 x 6



Empliq Amnio-Lite

Lyophilized Amniotic Membrane

Code	Size (cm) (length x width)
EM-AL2020	2 x 2
EM-AL4040	4 x 4
EM-AL4060	4 x 6

Available Through

ReNerve Ltd
7625 Golden Triangle Dr
Suite G
Eden Prairie, MN 55344



Legal Manufacturer

Berkeley Biologics LLC
880 Harbour Way South
Suite 100
Richmond, California, 94804

Contact ReNerve Ltd

Visit: www.renerve.com.au
Email: info@renerve.com.au
Phone: +1.612.293.8673

References

- (1) Data on File; Berkeley Biologics.
- (2) Greer, N.; et al. Ann. Intern. Med. 2013, 159 (8), 532–542.

Potential adverse events that may result from placement of Empliq products include but are not limited to wound or systemic infection, seroma, dehiscence, hypersensitivity, allergic or immune response, sloughing, graft failure, or disease transmission. Physicians should review the package insert for product information, including indications, contraindications, precautions, storage and handling instructions, and warnings.