

ReNerve<sup>LTD</sup>



**Lyophilized  
Amniotic Allografts**

**Protective barrier for  
complex wounds**



Patients with non-healing wounds face challenges that limit healing, leading to chronic defects and increasing risk of morbidity, amputation, and mortality.<sup>1</sup> Ongoing care for non-healing wounds also has an economic cost estimated in the billions annually.<sup>2</sup>

Empliq™ amniotic allografts act as a barrier and protect the wound bed during acute and chronic wound closures.

## The Empliq Range

Empliq allografts are intended for homologous use as a barrier, forming a protective cover over acute and chronic wounds.




### Empliq Placental

Proprietary process retaining chorion and amnion



### Empliq Amnio-Lite

Amnion only, no chorion

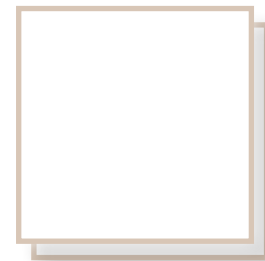

### Empliq Amnio-Duo

A double-layered amniotic membrane with improved handling characteristics

**Empliq Amnio-Lite** and **Empliq Placental** have two distinct sides: an epithelial side and a stromal side. The epithelial side is smooth while the stromal side is dull.

No orientation requirements, both sides are identical.



Retains native components of amniotic membrane



Easy to handle, orient, and apply



Terminally sterilized



No rehydration required



Shelf-stable



Multiple size options

## Application Guide

1

Select appropriate size and remove from packaging.

2

Apply stromal-side down against the wound bed, and trim excess away or fit to the contours of the wound as required.  
*(No specific orientation is required for Empliq Amnio-Duo.)*

3

Cover with a non-adherent dressing, followed by moistened gauze to fill, but not pack the wound.

4

Change dressings and reapply as clinically indicated.



ReNerve<sup>LTD</sup>



# Ordering Information

Lyophilized Amniotic Allografts



## Empliq Placental

Lyophilized Placental Membrane

Code	Size (cm) (length x width)
EM-PL2020	2 x 2
EM-PL4040	4 x 4
EM-PL4060	4 x 6



## Empliq Amnio-Duo

Lyophilized Double Amniotic Membrane

Code	Size (cm) (length x width)
EM-AD2020	2 x 2
EM-AD4040	4 x 4
EM-AD4060	4 x 6



## Empliq Amnio-Lite

Lyophilized Amniotic Membrane

Code	Size (cm) (length x width)
EM-AL2020	2 x 2
EM-AL4040	4 x 4
EM-AL4060	4 x 6

### Available Through

ReNerve Ltd  
7625 Golden Triangle Dr  
Suite G  
Eden Prairie, MN 55344



### Legal Manufacturer

Berkeley Biologics LLC  
880 Harbour Way South  
Suite 100  
Richmond, California, 94804

### Contact ReNerve Ltd

**Visit:** [www.renerve.com.au](http://www.renerve.com.au)  
**Email:** [info@renerve.com.au](mailto:info@renerve.com.au)  
**Phone:** +1.612.293.8673

#### References

- (1) Data on File; Berkeley Biologics.
- (2) Greer, N.; et al. Ann. Intern. Med. 2013, 159 (8), 532–542.

Potential adverse events that may result from placement of Empliq products include but are not limited to wound or systemic infection, seroma, dehiscence, hypersensitivity, allergic or immune response, sloughing, graft failure, or disease transmission. Physicians should review the package insert for product information, including indications, contraindications, precautions, storage and handling instructions, and warnings.